

Vista™ basic

Infusion Pump



Introducing the Vista™ basic Infusion Pump. Based on a design that has sold over 80,000 units worldwide, the Vista basic provides high reliability packaged in one of the easiest pumps to use.

B | BRAUN

B. Braun understands that alternate site clinicians and patients need a pump that is cost-effective and simple to operate and has the ability to do more than continuous infusions. The Vista™ basic was designed to ensure simplicity in all aspects of its use: from programming, to set installation, to alarm correction and special mode use.

SPECIFICATIONS

Mechanism	Linear peristaltic
Width	5.5 in (14 cm)
Height	9.4 in (24 cm)
Depth	7.8 in (20 cm)
Weight	6.8 lbs (3.1 kg)
Power	100-120 VAC, 50-60 Hz, 13 Watts
Leakage Current	max 30µA
Battery	7.2V, 1.2Ah NiCd (rechargeable)
Battery operating time	>3.5 hours at max. delivery rate
Rate Range	0.1 to 800.0 mL/hr in 0.1 increments with Vista™ tubing
Volume to be Infused Range	0.1 to 9999.9 mL
Occlusion Pressure	Variable settings: Low: 8 PSI and High: 17 PSI
KVO Rate	1 mL/hr or selected rate, whichever is lower
Flow Rate Accuracy	± 5% measurement time 8h, temperature 71.6 degrees F
Alarms	Air-in-Line, Low Battery, Plug In Now, Bag Empty, Downstream Occlusion, Door Open, Pump Stopped, KVO, Unit Malfunction
Memory	User option to retain last Continuous and Piggyback Mode data upon power up. Permanent data retention in Program and Ramp/Taper Modes after the pump is turned off.
Tubing	B. Braun Vista™ administration sets
Modes/Features	Continuous and Piggyback Modes, Program Mode for variable infusions, Ramp/Taper Mode for TPN infusions, and Delay A Mode for postponing infusion start and finish.

NOTHING COULD BE EASIER

- **Straight-line tubing** loads quickly into the pump.
- Pump-based **free-flow protection** prevents accidental boluses when door is opened.
- For continuous infusions, just enter the rate and volume, then press START.
- **Helpful prompts** walk the user through the programming process.

PROGRAM MODE

- **Nine steps** are available for rate and volume data entry.
- Sequential step delivery is **ideal for circadian and intermittent therapies**.

RAMP/TAPER MODE

- Step by step data entry with prompts makes programming **automatic ramp/taper infusions** simple.
- Maximum plateau rate is calculated by the pump and must be approved by the user for **added patient safety**.

OPERATING CONVENIENCE

- **Oversized display** allows the user to see information at a glance.
- **Lightweight components** and ergonomic handle placement on a **sturdy chassis** make pump transport easy under difficult conditions.
- Handy bag hook allows smaller bags (≤1L) to hang from the pump, with **no need for an IV pole**.



ORDERING INFORMATION

To order, call 800-227-2862.

Description	Catalog No.	Rx only
Vista basic Infusion Pump	637-102	See <i>Instructions for Use</i> for complete directions for use. Vista is a trademark of B. Braun Medical Inc.